

## **RAMP KIT 104 PLATFORM**

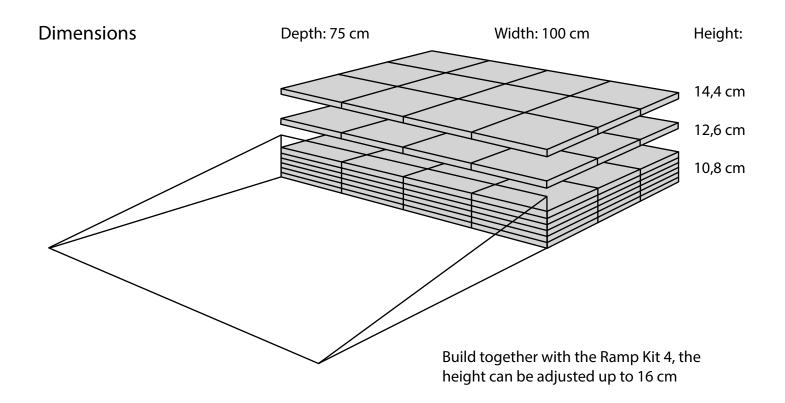


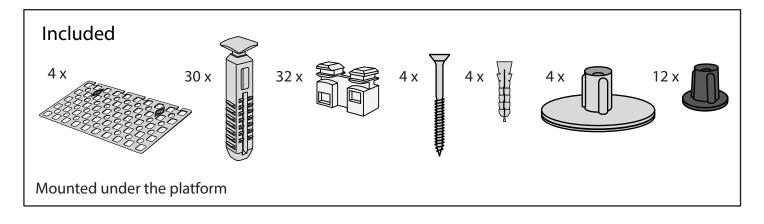









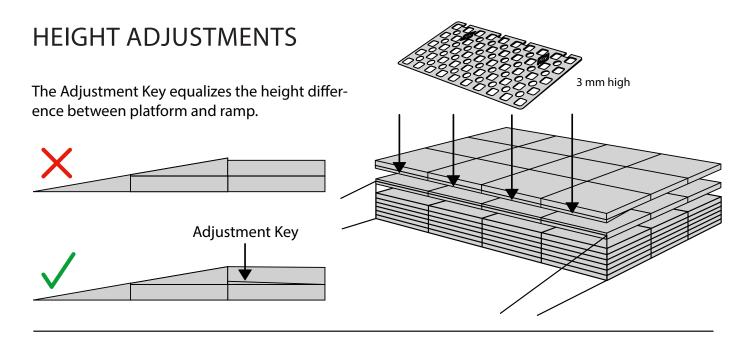


When you connect a platform and a ramp, remove the Ramp Adjuster from the ramp, and use it to adjust the platform as shown here:

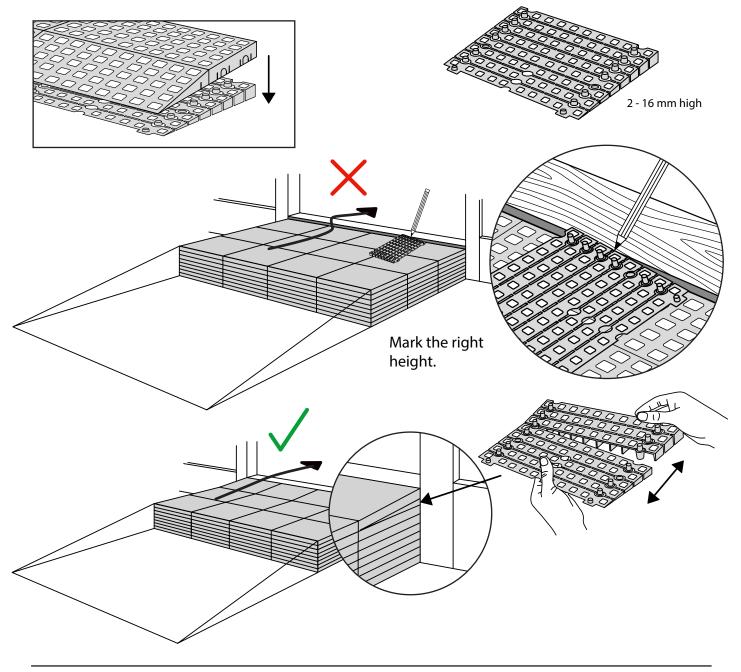


## PLACEMENT OF LOCKS

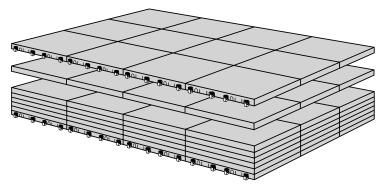

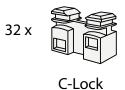


How to connect KIT and Platform:

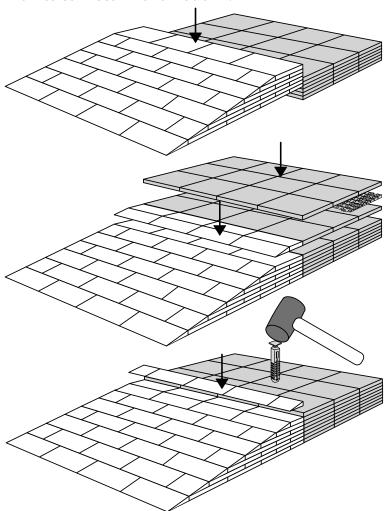


Locks in Ramp KIT: See KIT instruction for placement of Locks in the Ramp KIT.

Locks in platform: S-Lock is mounted on the top layer.


Use rubber hammer (Not included)

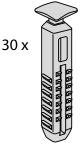


S-Lock

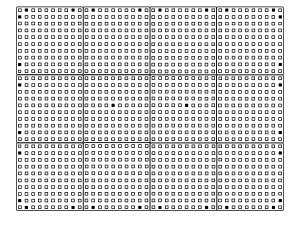

## **MOUNTING WITH STOPPERS**

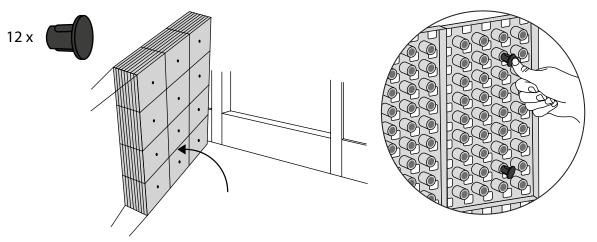

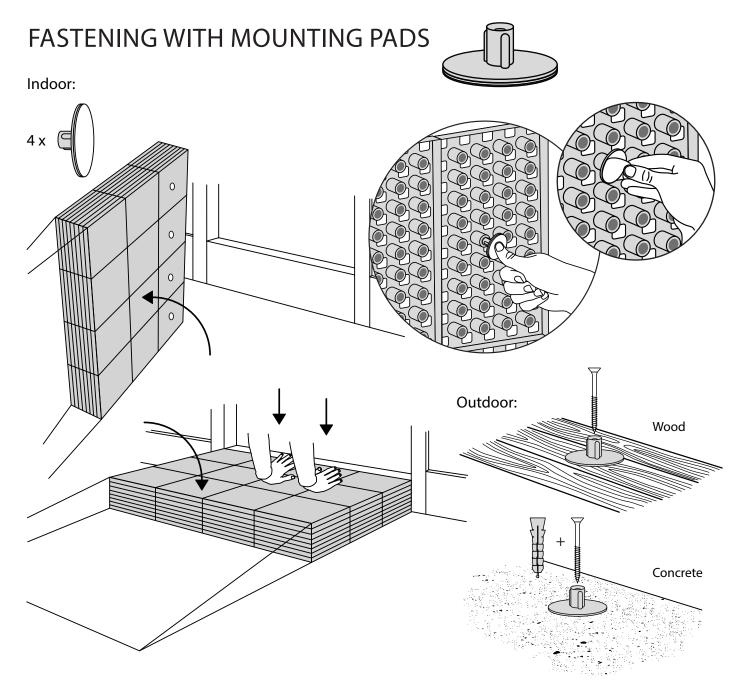

## **DISPOSAL**



KIT and KIT parts: These products are made from PE (Polyethylen). For disposal after use we encourage you to make sure that the KIT is either returned to the point of purchase for collection and shipment back to Excellent (for regeneration), or sorted accordingy to the recommendations in your local area.



Plastic bag: Recycled as clear LDPE plastic.



Cardboard box: Recycled as cardboard.



Brochure and instruction: Recycled as paper after end use.

