
PREASSEMBLED

RAMP AND SHOWER SOLUTIONS


KIT PACKAGES COMES PREASSEMBLED AND READY TO INSTALL, EASY TO ADJUST IN HEIGHT FROM 18 - 199 MM




The KIT system is easy to install and adjust in height. There are many ways of combining ramps, platforms and corners for the best access. Different surfaces are available for indoor and outdoor solutions.

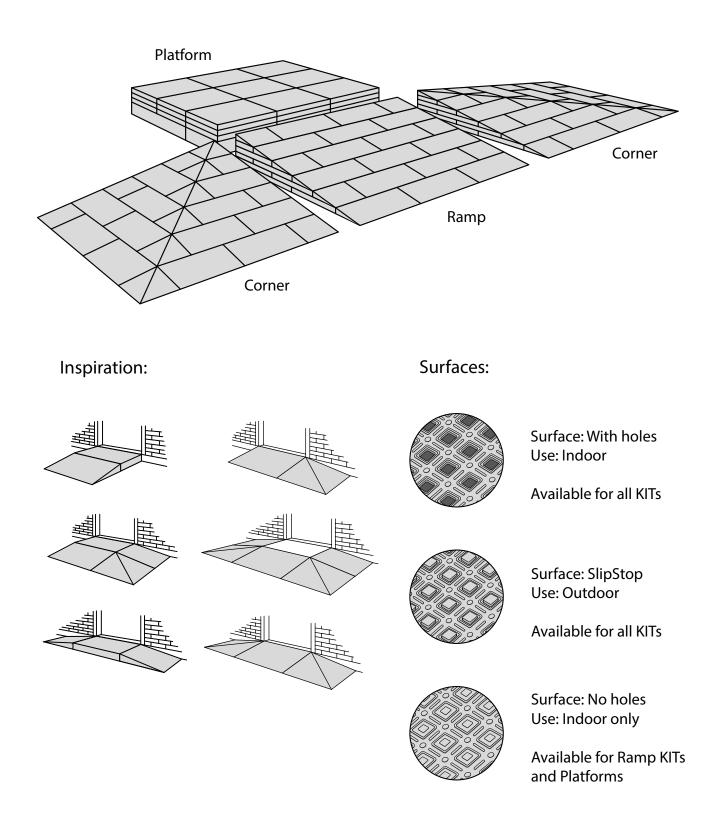


The two upper layers of the KITs are loose to make it possible to do the final adjustments in different locations and different kind of thresholds. Use the Ramp Adjuster to adjust the height.

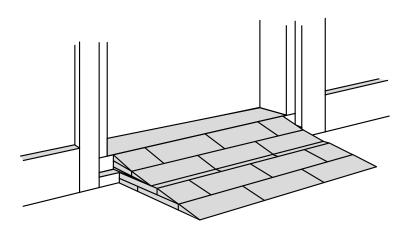


Different combinations of Ramp KITs on a threshold:

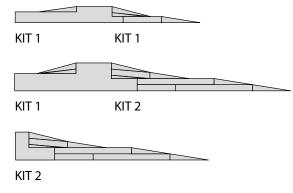

Sparepart packages:

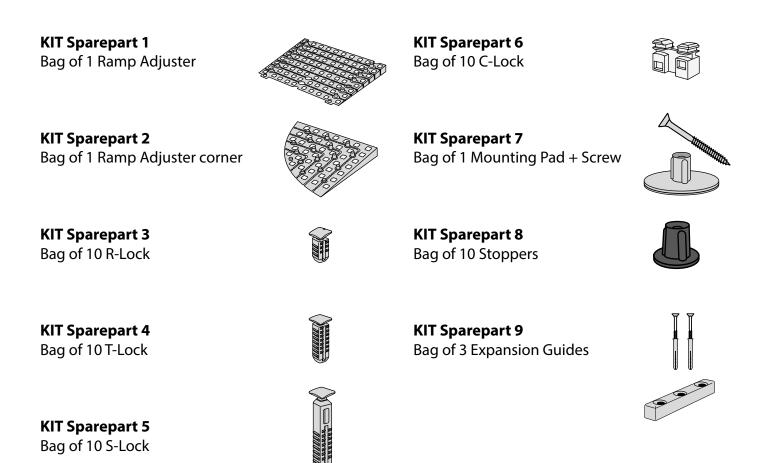


RAMP KITS

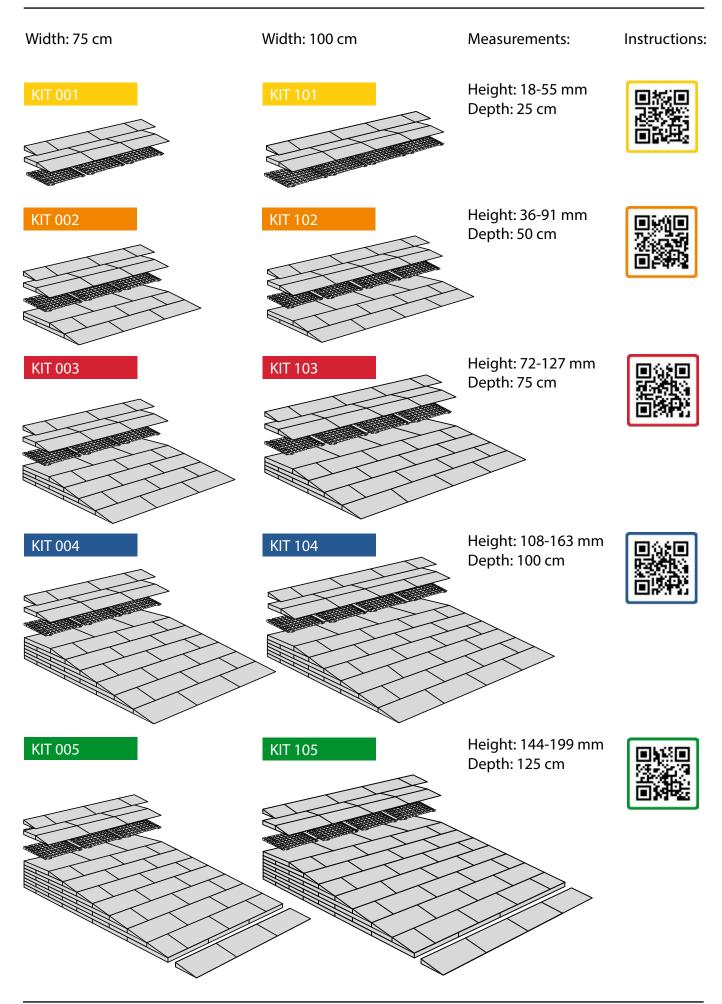

PLATFORMS

CORNERS

Width: 75 / 100 cm Depth: 75 cm

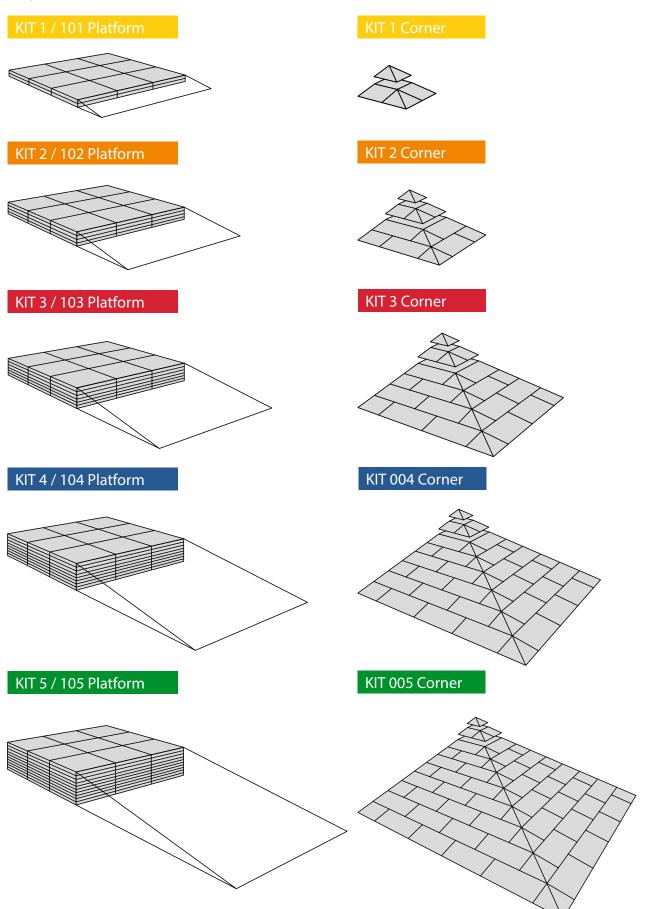

SHOWER KITS

Getting in and out of the shower can represent a great obstacle for many people because of the need for either a raised edge or a recessed area to contain the water. This creates an obstacle, both for the user and for those assisting the user in the shower.

By using Excellent Systems well proven ramps and tiles, it is quick, easy and above all, affordable, to overcome these obstacles by leveling the floor to the correct height while maintaining slip resistance and water drainage through the holes in the tiles.

It is safe to use Shower KITs with heavy wheelchairs and shower commodes. You might also need our Ramp Kit to assist with the entrance into a shower area.

How to use the shower KITs

Use Shower KIT 1 if the recessed area fits the height of one tile.

Edge to the floor 18 mm

Use Shower KIT 2 or 3, if the shower recess area is more than 2 mm lower than the tiles or tray edge.


Edge to the floor 12 mm


Use KIT 3 if the edge is more than 18 mm and you need two layers of tiles.

Edge to the floor 30 mm

16 tiles 100x100 cm Shower KIT 2 16 tiles 100x100 cm and 4 ramps Shower KIT 3 16 tiles 100x100 cm 8 ramps 1 corner ramp


Shower KIT examples

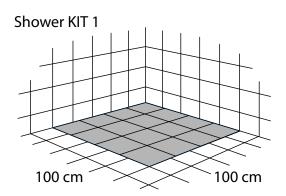

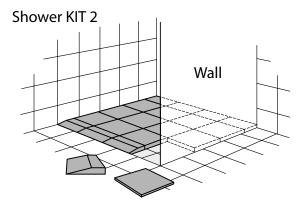

Shower KIT 3


Shower KIT two layers and a KIT ramp

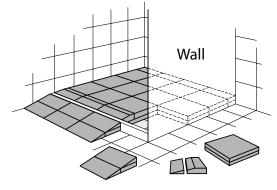


Cleaning of Shower KIT

The surface of the tiles and ramps have no pores. And thus cannot absorb bacteria. Water and soap can easily exit, through the holes in the tiles.

We recommend cleaning the tiles every four weeks in ordinary use. The tiles tolerate all types of conventional cleaning agents. They also tolerate autoclaving (+130° C) to be sterilized.

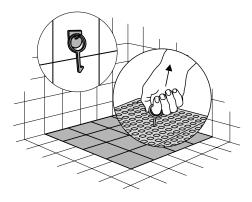

"Shower hook" makes it easy to lift the tiles in the shower.

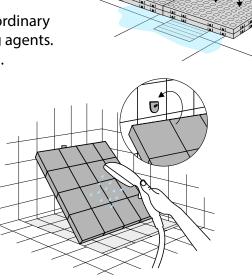

"Cleaning support" mounted on the wall, support the tiles while cleaning.





Excellent Systems A/S Møllevej 2, DK-8544 Mørke sales@ex-as.com +45 86377133 excellent-ramp.com Danish Design Pat. Pend. and produced in Denmark by Excellent Systems A/S

Brochure 2024